AKUS™ ANAESTHESIA NEEDLE FOR SINGLE USE

Whitacre, Sprotte or Quincke spinal needles, with tri-bevelled introducer.

IINSTRUCTIONS FOR USE FOR WHITACRE, SPROTTE OR QUINCKE TYPE SPINAL NEEDLES

Indications:

Spinal needles designed for performing spinal anaesthesia and lumbar puncture for extraction of cerebrospinal fluid. The Whitacre and Sprotte spinal needles are called "pencil point", and are designed to minimize lesions of the dura mater, reducing the risk of loss of cerebrospinal fluid, which is the cause of PDPH (Post Dural Punction Headache) – post lumbar punction headache.

The Quincke type spinal needle has a bevel type tip and its use is recommended in lumbar punctures for the extraction of cerebrospinal fluid (CSF).

The use of these types of needles is restricted to medical personnel with the training and appropriate experience necessary to undertake spinal anaesthesia and lumbar puncture techniques.

Instructions for use:

- Inform the patient about the technique being used and obtain written consent before starting the procedure.
- Closely inspect the proposed site for infection and lesions before performing the puncture.
- If tattoos are present in this area, assess and explain the risks to the patient.
- Correctly prepare the puncture site and cover it with a sterile field prior to starting the technique.
- Inspect the puncture material closely, ensuring that it has been packaged correctly and the needle is sterile. If not in perfect condition or sterility has been compromised, discard immediately.
- The AKUS™ spinal needles come with an introducer. The introducer has a 15° tri-bevelled tip which aids insertion through the skin. It therefore avoids the need to raise a subcutaneous wheal with local anaesthetic. The use of this introducer allows the later introduction of the needle and avoids dragging epidemial cells into the intracranidhan space.
- The AKUS™ needle's anatomic grip facilitates its accurate insertion, as bone, intramuscular and cartilaginous structures can readily be detected. This enables correct positioning in the subarachnoid space and avoids tip deformations. In the case of contact with bone and tip damage, withdraw the needle immediately and use a new one.
- Once in the subarachnoid space, withdraw the stylet and wait for CSF flow-back. Immediate visualisation of CSF flow-back is facilitated by the magnifying hub® in the AKUS™ transparent pavilion. This feature is unique to this needle.
- The magnifying hub® and the needle's body size, allow observation of a normal CSF. If the CSF is not transparent and appears turbid or blood stained, consider aborting the technique or puncturing in a different medullary space.
- The AKUS™ needle can be used with LUER cones and LUER-Lock syringes. Once the syringes filled with local anaesthetic, gently press and insert it into the intrathecal space.
- If desired, aspirate the syringe slightly to prove that the needle is correctly positioned in the intrathecal space, "barbotage technique".
- Likewise, the AKUS™ needle's front extension allows greater control and grip with your fingers, preventing antero-posterior movements that could cause technique errors.
- Withdraw the needle and the introducer, disinfect the skin and place a sterile dressing.

Contraindications:

- Absolute:
 - Patient's refusal of the technique.
 - Serious coagulation disorders, anti-coagulant treatment,
 - Skin infection in the place of puncton.
 - Septicaemia
 - Allergy to local anaesthetics.
 - Severe intracranial hypertension.
- Relative:
 - Anatomic anomalies
 - Skin tattoos.
 - Skin infections away from puncture site.
 - Mild coagulations disorders.
 - Non-corrected blood volume deficit
 - Dehydration.

Technique complications:

- PDPH (Post Dural Punction Headache) – post lumbar punction headache
- Hypotension
- Urinary retention
- Lumbago
- Paraesthesia
- Partial paralysis
- Total paralysis
- Subdural or epidural haematoma
- Meningitis
- Systemic toxic reaction
- Extensive block
- Infection (epidural abscess)
- Respiratory depression
- Cardiac arrest
- Death

Warning:

- All AKUS™ spinal needles have been sterilized by ethylene oxide.
- Single use needle; must not be re-used.
- The AKUS™ needle is phthalate-free.
- Do not use if the packaging has been damaged.
- Ensure that the necessary material for general anaesthesia and cardiopulmonary resuscitation is readily available.

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Jiangsu Province Huaxing Medical Apparatus Industry Co., Ltd.,